

CHAISE PAD REMOVAL

SOME STYLES HAVE A REMOVABLE CHAISE PAD
FOR CLEANING OR ADJUSTMENT PURPOSES
*NOT APPLICABLE TO ALL STYLES

1. Open lift chair to TV position
2. Detach chaise pad from velcro strip at the back of seat and ottoman
3. Remove chaise pad

FABRIC CARE

We use only the finest upholstery materials because the beauty of your furniture depends on its cover. See marked cleaning codes & instructions below:

WS

Water based cleaning agents and foam may be used for cleaning. This fabric may also be cleaned with mild, water-free dry cleaning solvents

W

Water based cleaning agents or water-based foam may be used for cleaning.

S

Only mild, pure, water-free dry cleaning solvents may be used for this fabric.

X

This fabric should be vacuumed or brushed lightly to remove soil. **WARNING:** Do not use foam or liquid cleaning agents of any type.

V

This fabric should be washed with warm water and mild soap. **WARNING:** Do not use cleaner or liquid cleaning agents of any type.

LIFT CHAIR OWNER'S MANUAL AND OPERATING INSTRUCTION

ATTENTION: Read owner's manual before attempting to use your new lift chair




COMFORTLIFT

Adds comfort and convenience to
your lifestyle

SET UP INSTRUCTIONS

1. Read through all instructions before beginning.
2. Remove all plastic wrap and cardboard shipping material.
3. Set aside lift chair back.
4. Locate plastic bag at back of seat unit. Bag contains chair motor wire, hand control wand, power cord and transformer.
5. Remove items from plastic bag. Discard plastic bag.
IMPORTANT: Insure chair motor wire remains under wooden back rail. Lay hand wand and cord up and over seat base and onto seat (as shown in photo 1) before attaching back. Secure hand wand wire in velcro (As shown in photo 2).
6. Place lift chair back on seat arms and fold flap back exposing connecting brackets, (as shown in photo 3)
7. Rotate chair back from front toward back of lift chair and line up back brackets with male seat brackets, (as shown in photo 4)
8. Align connecting brackets for lift chair seat and back and briskly press down on chair back until brackets "click" together, (as shown in photo 5)
IMPORTANT - Make certain that back is locked down properly - you should not be able to remove back. Failure to properly install back could cause serious injury.
9. Plug in transformer power cord to electrical wall outlet.
Note: Do not use an extension cord. Plug directly into wall outlet.
10. Using hand wand, raise chair into full upright or standing position. Connect velcro at bottom edge of back flap with velcro on bottom edge of lift chair frame (as shown in photo 6)

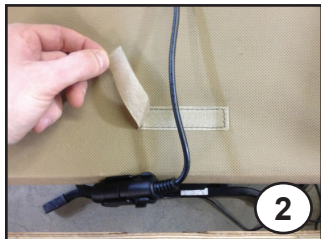

THREE-YEAR LIMITED WARRANTY

FRAME / MOTOR / MECHANISM / HAND CONTROL

Warranted to the original owner/purchaser to be free of defects in workmanship and materials for a period of three (3) years from the date of purchase. All other components of the COMFORTLIFT chair are warranted to be free of defects in workmanship for one year from the date of sale to the original owner/purchaser.

LIMITATIONS ON WARRANTY

The warranty contained herein shall be in lieu of any other warranty, expressed or implied, including but not limited to any implied warranty of merchantability or fitness for a particular use. Liability for incidental or consequential damages under the warranty is excluded. Some states do not allow limitations of incidental or consequential damages; therefore this limitation may not apply to you. This warranty give you specific legal rights and you may also have other rights that vary from state to state. The following additional limitations apply to all warranties contained herein:

1. Warranties extend to the original purchaser only and are not transferable.
2. Warranties do not apply to rental, business, commercial, institutional, or other non-residential use.
3. All warranties are subject to the availability of repair and replacement parts and are limited to a one-time replacement. Shipping and handling costs are not included.
4. Replacement cover dye lots vary from shipment to shipment and may not exactly match the original.
5. Warranties do not apply to any conditions caused by damage, improper use or abuse occurring after the date of sale to the original purchaser. Such damage, improper use or abuse will void all warranties.

FRANKLIN'S OBLIGATION

Franklin's obligation under this warranty is limited to repairing or replacing at Franklin's discretion, any part or parts of the chair covered by warranty that are found to be defective in workmanship or materials upon examination by Franklin, or a Franklin designated agent. Cost of packing and shipping the entire piece to and from the factory, if necessary, is not covered by this warranty. Any merchandise deemed not repairable by the manufacturer will be prorated through the dealer based on the original date of purchase.

HOW TO OBTAIN WARRANTY BENEFITS

To obtain benefits under the warranty, contact the dealer from whom you purchased this merchandise. If you move away from the area serviced by your dealer or if your dealer is no longer in business, you may write:

FRANKLIN CORPORATION, P.O. DRAWER 569, HOUSTON MS 38851

or visit our website at www.franklincorp.com

Franklin reserves the right to require that defective parts be returned to the factory upon request. Proof of purchase is required in order to obtain benefits under the warranty.

BATTERY BACKUP

Your COMFORTLIFT chair is equipped with a battery back-up that will automatically activate during power failure. This system is designed to take you from full recline to the closed position. Your battery back-up system is powered by two 9-volt batteries (not included). The two battery terminals are located in the external transformer. Always make sure your lift chair is equipped with two fresh batteries in case of power failure. The batteries need to be replaced every time the back-up system is used or once a year if the battery back-up system has not been used. To install batteries, remove trap door on external transformer with a screwdriver, or push forward with your fingers (bottom chamber type) or with the spring clip (top chamber type). Snap battery terminals to the batteries and insert into the chamber. Reattach the door.

COMFORTLIFT chairs with an independent motor for the backrest and an independent motor for the footrest have a weight capacity of 500lbs. All other lift chairs will have a 350lbs. weight capacity. Exceeding this will void your warranty and could cause damage to the lift and harm to the operator.

Care and Maintenance

To insure your COMFORTLIFT chair allows maximum comfort expectation and offers years of trouble-free operation, some routine maintenance and inspection will be required. Inspect the fabric on a regular basis for pulls, tears, or gaps. Also, keep fabric clean using the cleaning instructions found in your operation manual.

Regulate the seat and back padding by shifting foam or fiber to help maintain a crisp and well-tailored look. Some lift chair styles are supplied with a full pad over removable chaise pad. This pad is attached with Velcro and can be removed for regulating, replacing, or cleaning (refer to the owner's manual for fabric cleaning instructions).

Visually inspect the power cord for any cracks or wear.

Periodically check the hand control cord for cracks, wear, holes and loose wires.

CAUTION: Choose the right methods to clean fabric on lift chair. Before cleaning, consult your authorized COMFORTLIFT provider for cleaning codes. DO not use solvent-type cleaners to spot clean. DO not saturate fabric.


Refer to fabric care instructions on page 8

11. Position your new lift chair in close proximity to a standard (110 AC) wall outlet which is free from obstacles. Make sure you allow enough space between the back and nearest obstruction so that you can easily recline. This clearance may vary from style to style due to different back heights used. Level your chair using the leveling glides on the metal base of the chair. These glides can be adjusted in and out by rotating clockwise to lower and counter clockwise to raise the base (4 per chair). Not all floors are perfectly flat and level. These glides will help compensate for the height difference found in some floors. After the chair is leveled, turn chair over and unwind power cords. Route the power cord so it is free from the metal base and other obstacles which could pinch or cut the cord. Plug the power cord into a standard (110 AC) outlet (DO NOT use an extension cord). For some international applications, a voltage converter may be required. Your new COMFORTLIFT chair works on a low voltage DC motor that reduces the standard household current (110 AC) to a 29v DC. The COMFORTLIFT lift mechanism is equipped with an internal thermal shutoff switch to prevent the motor control box from overheating. If the motor stops during a lifting cycle, allow the lift chair to remain in this position for 10 minutes. This will allow the motor control box to cool and reset itself. Normal use can resume after cooling. If the problem continues, contact your authorized COMFORTLIFT dealer for assistance.



7

Diagram #1

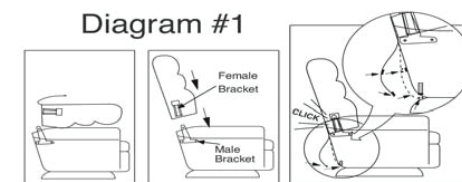

WARNING!

Do not allow food particles and/or liquids (for example sodas) to fall into the hand control switch opening. Failure to do so may impair the functionality of the switch motion.

If you notice severe wear on hand control cord and/or hand control and if hand control does not function properly, contact your authorized COMFORTLIFT provider.

CAUTION!

If you notice damage to the wires leading to massage units, unplug your lift chair immediately and call your authorized COMFORTLIFT provider for service. Otherwise, serious personal injury may result.

1. Do not allow children to play on this mechanized furniture or operate the mechanism.
2. Leg rest folds down on closing so that a child could possibly be injured.
3. Always leave in an upright and closed position. Keep hands and feet clear of mechanism and all moving parts. Leg rest must be completely closed before getting in or out of the chair. Only the occupant should operate the chair.
4. Never sit on the leg rest when open.
5. Do not sit on the arms of the recliner.
6. Use caution in operating the reclining lift mechanism when small children and pets are present.
7. Completely close and secure the reclining mechanism before moving motion furniture.
8. Avoid exposing upholstered furniture to burning cigarettes or cigars, open flames and high temperatures.
9. Do not exceed recommended weight limit.

IMPORTANT SAFETY INSTRUCTIONS

WARNING: To reduce risk of injury to persons:
READ INSTRUCTIONS BEFORE OPERATING.
INJURY MAY RESULT FROM IMPROPER USE.
NEVER REMOVE COVERS.

- This lift chair should never be left unattended when in operation.
- Unplug lift chair from outlet before relocating or troubleshooting.
- Do not connect an extension cord to this unit
- Close supervision is necessary when this lift chair is used by, on, or near children, invalids, or disabled persons.
- Use this lift only for its intended use as described in the operating instructions.
- Do not use attachments not recommended by the manufacturer.
- Keep all cords away from heated surfaces and pinch points.
- Do not use outdoors. INDOOR USE ONLY.
- To disconnect, turn all controls to the off position, then remove plug from outlet.

DANGER - TO REDUCE RISK OF ELECTRIC SHOCK:

- Never use pins or other metallic fasteners with this lift chair.
- Discard the lift chair if the covering shows any sign of deterioration, such as checking, blistering, or cracking.
- Keep dry. Do not operate in a wet or moist environment.